
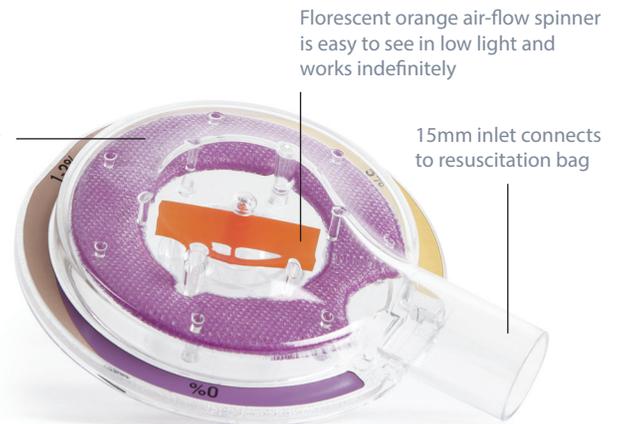


# FLOCAP CO<sub>2</sub> COLORIMETRIC + FLOW INDICATOR

LATEX FREE | SINGLE USE | DISPOSABLE

The FloCap is a CO<sub>2</sub> colorimetric indicator designed to visualize a patient's exhaled CO<sub>2</sub> and expiratory flow. The CO<sub>2</sub> colorimetric display can help caregivers verify proper endotracheal tube placement. The patent-pending spinner detects end of exhalation, which can prevent breath stacking and associated health implications during resuscitation.

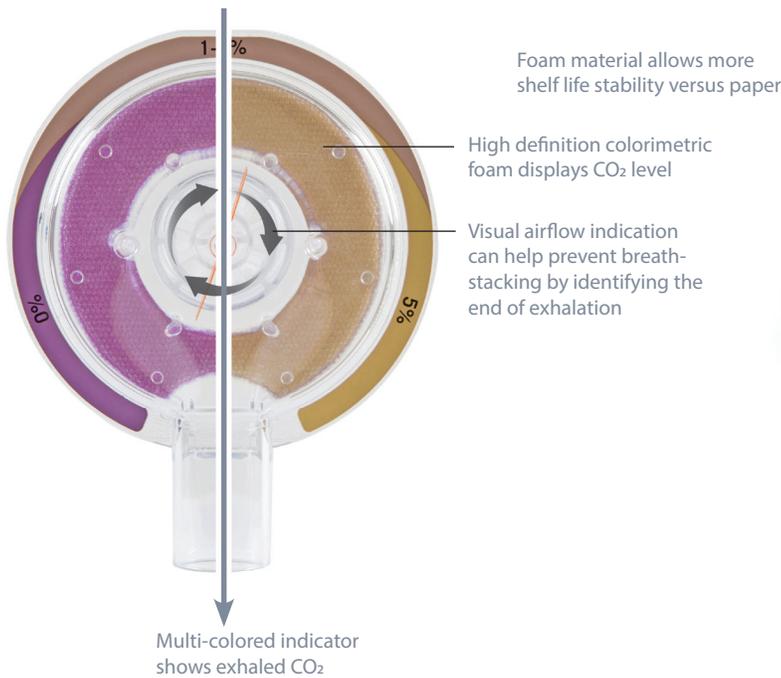

### COLOR & FLOW INDICATOR INTERPRETATION

Purple (0%)	No carbon dioxide detected
Beige (1-2%)	Exposed to CO <sub>2</sub> , approx. 1% to 2%
Yellow (5%)	Exposed to 5% or greater CO <sub>2</sub>
Spinning*	Patient is exhaling
Not Spinning	Exhalation flow < 3 LPM

\*At the end of exhalation, the flow indicator's momentum will cause it to spin briefly after exhalation has ceased.

## FLOCAP CO<sub>2</sub> COLORIMETRIC + FLOW INDICATOR

- 24 hour use duration - from room-to-room and during transport
- Fully compatible with all SunMed, adult-sized resuscitation devices
- For patients greater than 15 Kg (33 lbs)

ITEM#	DESCRIPTION	PK
R500P21	CO <sub>2</sub> Colorimetric and Flow Indicator	24



MSM-REF-0110 Rev. 1